

### Supplies

1. New neckband
2. Mirror
3. Scissors



Detach the wide VELCRO® tab from neckband to separate into two pieces.



Place the two sections in front of you and detach the narrow VELCRO® fasteners from neckband.



Secure the neck flange while removing one side of the old neckband.



Take the short section and thread with the VELCRO® side facing out (smooth side against neck). Attach fastener to the neckband.



Secure the neck flange while removing the other side of the old neckband.




Take the long section and thread with the VELCRO® side facing out (smooth side against neck). Attach fastener to the neckband.



Attach the wide VELCRO® tab of the short section to the long section. Attach fastener to the neckband.



Insert two fingers under the neckband to ensure fit.

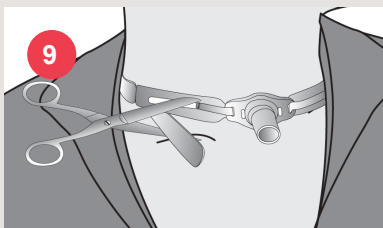

Trim excess neckband material with scissors, if needed. Discard the old neckband.



### Tip #1

Changing of the neckband is a two person activity. You can secure the neck flange while your helper removes the old neckband and applies new neckband per instructions or vice versa.

### Tip #2

Once the old neckband is removed, it is important to assess the skin under the neckband before securing new neckband in place. Pressure induced trauma can occur if neckband is worn too tightly. Inspect skin for signs of infection, irritation, and skin breakdown.

### Tip #3

Change neckband every 24 hours or more often as needed.

#### Head office:

Atos Medical (UK)  
Cartwright House,  
Riverside Business Park,  
Tottle Road, Nottingham,  
NG2 1RT  
Tel: 0800 783 1659  
Text: 0753 741 7928  
Email: [info@atos-care.co.uk](mailto:info@atos-care.co.uk)  
Web: [www.atos-care.co.uk](http://www.atos-care.co.uk)

#### Manufacturer:

Atos Medical AB  
Kraftgatan 8  
SE-242 35 Hörby, Sweden

#### Manufacturer:

Tracoe Medical GmbH  
Reichelsheimer Straße 1/3  
55268 Nieder-Olm, Germany