

Supplies

1. Bowl
2. Swabs
3. Gauze dressing

Inner cannulas should be cleaned immediately after each inner cannula change.



1 Prepare a bowl with tap water (drinkable), or if needed, sterile or distilled water.



2 Rinse your used inner cannula in a bowl of luke warm water.



3 Clean the inner cannula from both ends with the swab. Use as many swabs as needed for cleaning.



4 Use a swab to clean the outside of the inner cannula.



5 Rinse your used inner cannula in a bowl of luke warm water.



6 Leave to air dry. Place in clean and dry storage container until next use. Discard swab after each use.

Attention

Never insert the swab into the tracheostoma itself to clean. The inner cannula must always be removed from the outer cannula prior to cleaning.



Head office:
Atos Medical (UK)
Cartwright House,
Riverside Business Park,
Tottle Road, Nottingham,
NG2 1RT
Tel: 0800 783 1659
Text: 0753 741 7928
Email: info@atos-care.co.uk
Web: www.atos-care.co.uk

Manufacturer:
Atos Medical AB
Kraftgatan 8
SE-242 35 Hörby, Sweden

Manufacturer:
Tracoe Medical GmbH
Reichelsheimer Straße 1/3
55268 Nieder-Olm, Germany

© Coloplast A/S. All rights reserved. Atos and the Atos Medical logo are trademarks of Coloplast A/S.

Please also note the applicable instructions for use. These instructions do not replace the IFU.

The word "Atos" in a bold, red, sans-serif font.