

Supplies


1. New HME



Perform hand hygiene.



Secure neck flange with one hand. Gently twist and pull to remove HME with the other hand. Discard used HME.

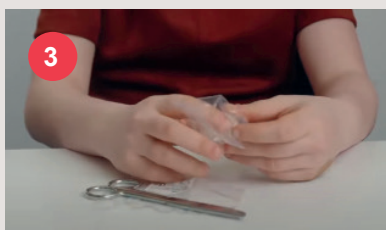

Take clean HME out of packaging.



Secure neck flange with one hand. Gently push and twist to attach HME with the other hand.

Attention

Replace your HME at least every 24 hours or more often as needed.

Tip #1

An HME (Heat and Moisture Exchanger) is a device which has been designed to work in a similar way to your nose by conditioning the air that you breathe. An HME fits on to the end of your tracheostomy tube, and you breathe in and out through it. The HME conditions the air you breathe by keeping it at a good humidity and temperature for your lungs to function properly.

Tip #2

Atos Medical supplies a variety of HMEs that are designed to suit different lifestyles and needs. These HMEs include special features such as the addition of a filter, oxygen connector, suction port, or speaking functions. Consult your healthcare provider to decide which HME is most appropriate for you.



Scan the QR code to watch the instructions as a video

Head office:
Atos Medical (UK)
Cartwright House,
Riverside Business Park,
Tottle Road, Nottingham,
NG2 1RT
Tel: 0800 783 1659
Text: 0753 741 7928
Email: info@atos-care.co.uk
Web: www.atos-care.co.uk

Manufacturer:
Atos Medical AB
Kraftgatan 8
SE-242 35 Hörby, Sweden

Manufacturer:
Tracoe Medical GmbH
Reichelsheimer Straße 1/3
55268 Nieder-Olm, Germany

© Coloplast A/S. All rights reserved. Atos and the Atos Medical logo are trademarks of Coloplast A/S.

Please also note the applicable instructions for use. These instructions do not replace the IFU.

The word "Atos" in a bold, red, sans-serif font.